

EN: Instructions for Use: Pedilay® foot

Product Description

Pedilay® foot is an adaptable, ready for use bandage for the foot that covers the foot up to the ankle.

The product is intended to be used on intact skin. The inside layer of the laminate consists of a non-woven material which causes low friction (shear forces) to the skin. Pedilay® foot can support the medical treatment of abnormally shaped feet (e.g., diabetic feet). This is realised using a two layered non-woven laminate with incorporated elastic strands. The hook and loop fasteners system is repositionable. Pedilay® foot can be used on the right or left foot.


Medical indication(s)

Adaptable, ready for use bandage for the foot: The product also supports the medical treatment of abnormally shaped feet (e.g., diabetic foot). For use as fixation bandages for universal use (also for wound pads). Dressing retention on the foot depending on size is possible. Use as protective bandage over wound pad material or plaster.

Method of application (recommended) or as directed by a doctor

APPLICATION STEPS

- 1.) Open the fasteners and put the Pedilay® foot over the toes.
- 2.) Pull Pedilay® foot back over the foot.
- 3.) Stretch the Pedilay® foot to the length of the foot and pull it up over the heel.
- 4.) Close all fasteners according to instructions.

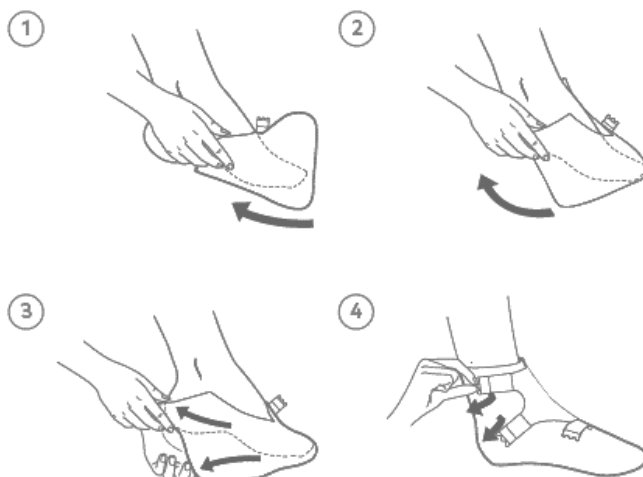

Additional information

It is recommended to pull on Pedilay® foot sitting or lying down.

Select the appropriate size of Pedilay® foot according to the size chart.

	M	L	XL
Shoe Sizes (EU)	37 – 42	41 – 46	45 – 49
Foot Length in cm:	20 – 26	25 – 32	28 – 35

The following factors may require a different size of Pedilay® foot:

- skeletal deformities
- extremely voluminous feet
- swollen feet
- amputations
- additional layers of primary/secondary layers

Pedilay® foot is correctly applied when no cavities form, there are no gaps or wrinkles, and the material does not apply pressure/compression.

Only use Pedilay® foot as described in the instructions for use. Any other use is considered as not intended and may lead to material damage and personal injury.

When using Pedilay® foot, make sure that the bandage is not applied too tightly, does not stretch the foot, and has sufficient elasticity on the foot.

If there is any uncertainty about the use or if the health condition worsens, the use should be immediately discontinued, and medical advice should be obtained. When walking with Pedilay® foot, it is recommended to wear adequate footwear (e.g., shoes, surgical shoes).

Please note: If, while using this product, you have any reason to assume that a serious event has occurred, please report the event to the manufacturer and your national authority.

Hygiene and Disposal

Change Pedilay® foot:

- daily with acute, infectious wounds
- whenever it is dirty
- whenever the material is stretched out or torn
- whenever the primary layer (wound dressing) underneath is changed

It is hazardous to re-use a medical device that is intended for single use. Recycling products in order to re-use them can have a seriously negative impact on their properties and performance.

Pedilay® foot can be disposed of with normal household waste.



Distributed by:

Distributor specification + contact information + website (if applicable)

 **KOB GmbH**
Lauterstraße 50, 67752 Wolfstein, Germany
www.kob.de

Date of revision: ww/yyyy