

EN: Instructions for Use Pedilay® leg

Product Description

Pedilay® leg is an adaptable, ready to use bandage for the lower leg.

The product is intended to be used on intact skin. The inside layer of the laminate consists of a non-woven material which causes low friction (shear forces) to the skin. Pedilay® leg can support the medical treatment of pathologically shaped lower legs (e.g., Ulcus Cruris). This is realised using a two layered non-woven laminate with incorporated elastic strands. The hook and loop fasteners system is repositionable. Pedilay® leg can be used on the right or left lower leg.

Medical indication(s)

Adaptable, ready for use bandage for the lower leg. The product also supports the medical treatment of pathologically shaped lower legs (e.g., Ulcus Cruris). For use as fixation bandages for universal use (also for wound pads). Dressing retention of the lower leg depending on size is possible. Use as protective bandage over wound pad material or plaster.

Method of application (recommended) or as directed by a doctor

APPLICATION STEPS


- 1.) Carefully pre-stretch the bandage at the lower end. Hold the seams to prevent tearing. Align the opening with the hook and loop fasteners system to the front.



- 2.) Carefully pull the lower end over the foot up to the ankle: Hold on to the straps of the opening close to the end of the seam: If necessary, the end can be spread open from the inside by hand to reduce friction on the foot.



- 3.) Pre-fix the closures one after the other, working from the bottom to the top. Make sure that the two sides of the opening overlap sufficiently. Then readjust the closures.

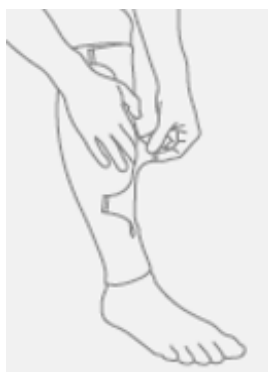

- 4.) Ensure a correct fit. The bandage should extend from the ankle to below the knee and fully cover the wound dressing.



- 5.) The bandage should fit smoothly and have no folds. Pedilay® leg must not be too taut or cut into the skin, and there must be sufficient residual elasticity.



- 6.) The bandage can be easily opened/closed to check the wound dressing. Regularly check the bandage to make sure it is still clean. If required, the closures can be tightened or loosened.



Readjust the closures if:

- there is a feeling of constriction
- the bandage is no longer elastic at one point or is very tight
- the bandage slips
- wrinkles appear in the bandage

Additional information


It is recommended to pull on Pedilay® leg sitting or lying down. Select the appropriate size of Pedilay® leg according to the size chart.

Available Size

The following must be observed when selecting a size:

- The thickness of the wound dressing
- The circumference of the lower leg can change over the course of the day due to oedema formation and reduction
- In the course of the healing process or following a change of treatment, the product size may have to be changed
- Other factors may require a different size in individual cases

Length from ankle to back of knee ℓB-D	Calf circumference cC	
Short	M	L
32–41 cm	28–42 cm	39–53 cm
Long		
39–48 cm		

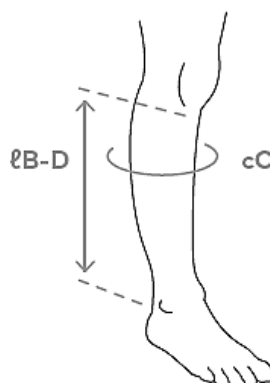

Pedilay® leg is correctly applied when no cavities form, there are no gaps or wrinkles, and the material does not apply pressure/compression.

Only use Pedilay® leg as described in the instructions for use. Any other use is considered as not intended and may lead to material damage and personal injury.

When using Pedilay® leg, make sure that the bandage is not applied too tightly and has sufficient elasticity on the lower leg.

If there is any uncertainty about the use or if the health condition worsens, the use should be immediately discontinued, and medical advice should be obtained.

Please note: **If, while using this product, you have any reason to assume that a serious event has occurred, please report the event to the manufacturer and your national authority.**

Hygiene and Disposal

Change Pedilay® leg:

- daily with acute, infectious wounds
- whenever it is dirty
- whenever the material is stretched out or torn
- whenever the primary layer (wound dressing) underneath is changed

It is hazardous to re-use a medical device that is intended for single use. Recycling products in order to re-use them can have a seriously negative impact on their properties and performance.

Pedilay® leg can be disposed of with normal household waste.



Distributed by:

Distributor specification + contact information + website (if applicable)

 **KOB GmbH**
Lauterstraße 50, 67752 Wolfstein, Germany
www.kob.de

Date of revision: ww/yyyy